

How to use Unistik® 2

1. Push



Push the lancet cap into the body of Unistik 2 until it clicks.

2. Twist



Twist off the lancet cap until you feel it separate from the device. Don't pull, just twist. Dispose of the cap in a suitable container.

3. Sample



Choose a site on the finger, or for neonates choose a site on the heel as indicated by the shaded areas on pictures A and B. Press the Unistik 2 platform firmly against the chosen site and press the release button.

If required, massage the sample site, taking care not to squeeze too hard at the site.

The single-use needle retracts immediately after sampling leaving the device safe for immediate disposal. Use a suitable sharps container.

The Unistik® 2 range

Unistik®2 Comfort

For frequent tests, such as blood glucose monitoring and for use in paediatric departments. The thin 28G needle and reduced penetration depth provide sufficient blood samples for more delicate skin.



Product codes: AT0740 Box of 50/AT0742 Box of 100

Unistik®2 Normal

For normal skin types and standard tests such as blood glucose monitoring in adults and cholesterol testing.



**Product codes: AT0700 Box of 50/AT0702 Box of 100/
AT0704 Box of 200**

Unistik®2 Extra

For use on tougher skin or for larger samples, e.g. haemoglobin and cholesterol.



**Product codes: AT0710 Box of 50/AT0712 Box of 100/
AT0714 Box of 200**

Unistik®2 Super

For even larger samples and multi-test situations. The specially ground needle provides more blood, for test areas such as blood gases and electrolytes, chemistry test profiles and other diagnostic kit tests requiring large volumes.



Product code: AT0752 Box of 100

Unistik®2 Neonatal

For heelstick tests on neonates, for test areas such as congenital hypothyroidism, phenylketonuria (PKU), sickle cell disease and other haemoglobin diseases.



Product code: AT0762 Box of 100